



SENSEN™

Balance is a Vital Sign.
Are you assessing every client?

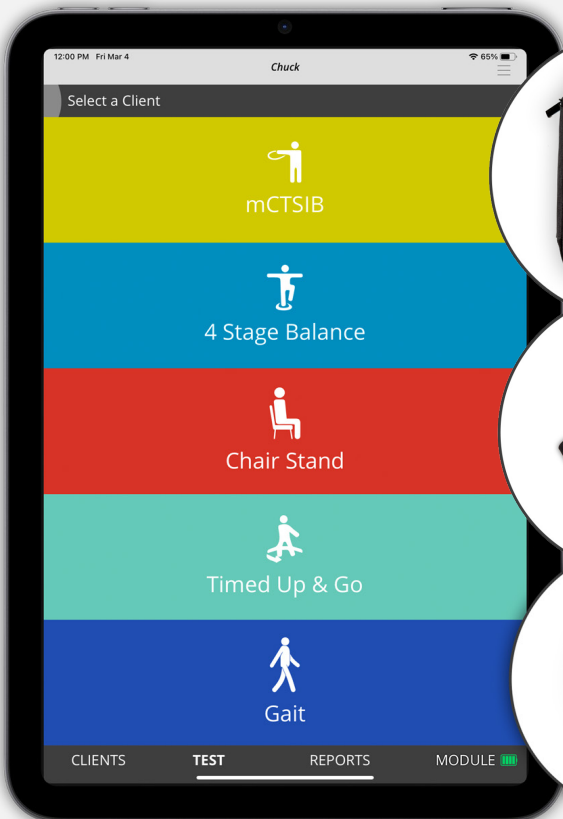

by
sen
METRIC

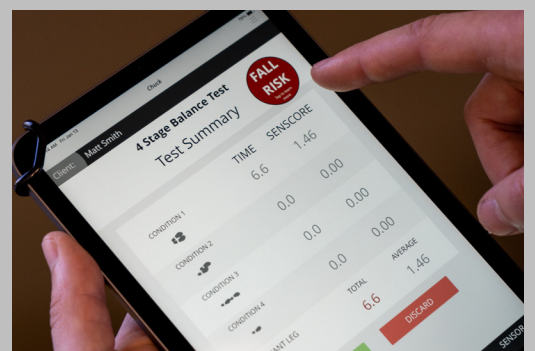

5 TESTS in under 12 MINUTES

4 Stage Balance Test

Assess Static Balance — an important part of postural sway

OVERVIEW

- Four, 10-second conditions with various foot placements
- Record a leg preference
- Objectively identify Fall Risk clients

OUTPUTS

- Sway Score - how much sway did the client exhibit during each condition, presented on a 1-10 scale
- Time in Balance - total time the client spent upright

Modified CTSIB

Assess how sensory inputs affect physical balance

OVERVIEW

- Four, 30-second conditions that strategically highlight vestibular, proprioception, and visual balance centers

OUTPUTS

- Sway Score - how much sway did the client exhibit during each condition, presented on a 1-10 scale
- Time in Balance - total time the client spent upright

30-second Chair Stand

Assess functional lower body strength and endurance

OVERVIEW

- Count total number of full stand-sit cycles
- Calculates a fatigue index - client's change in speed from beginning to end
- Objectively identify Fall Risk clients

OUTPUTS

- Total stand count (full reps only)
- Response time - how long the client took to begin test
- Fatigue Index - the change in time to stand from first to last rep

Gait Test

Assess functional mobility — and see stride length, step count

OVERVIEW

- Set custom distance
- Specify an acceleration start or start from standing still

OUTPUTS

- Gait Speed
- Stride Length, Step Count

Timed Up and Go

Simple performance measure of lower extremity function, balance, mobility, and fall risk

OVERVIEW

- Time a client as they walk out ten feet, turn around, walk back, and sit down
- Objectively identify Fall Risk clients

OUTPUTS

- Times broken down by section: walk out, turn time, walk back, total time
- Response time - how long the client took to begin test

REPORTING



SENSEN includes built-in reporting, designed to monitor client progress and support billing reimbursements.

Generate and share a variety of reports at the touch of a button, directly from the tablet, including custom .CSV exports for EHR uploading.




OUTPUTS & SCORING

SenSen shines with objective and repeatable scoring. In addition to traditional outputs (time, total count) SenSen enhances each test with a Sway score, detailed performance breakdown, reaction times, and a fatigue index — all possible with the SenSen's wireless sensor and Comfort Belt.

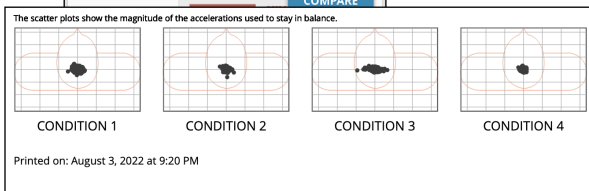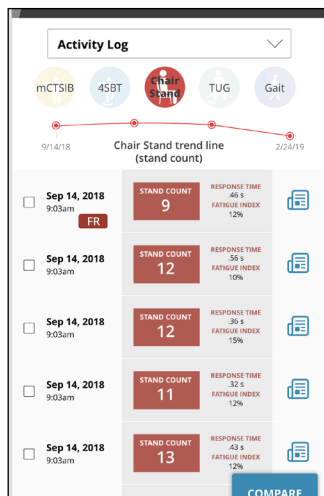

QUICK, EASY TO USE SOFTWARE

SenSen requires minimal training for any employee to use. Administer objective, consistent tests with ease.

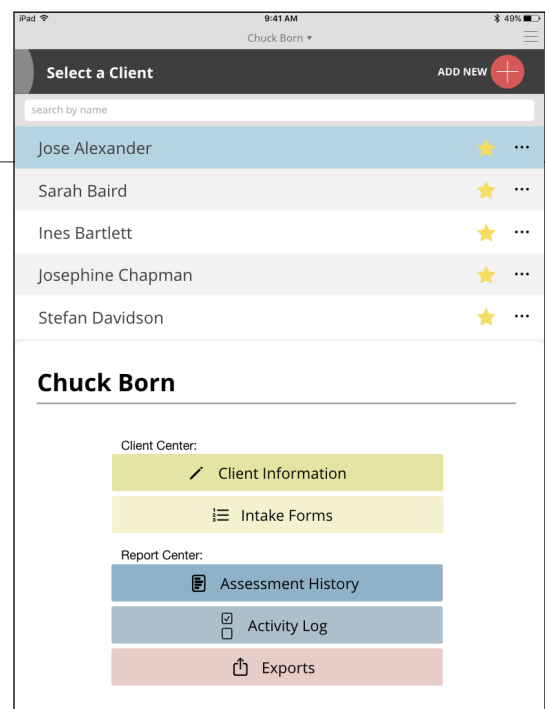


What is your Balance Assessment solution?

Over 1 in 4 older people fall
every year.

Less than half tell their doctor.

FR = Fall Risk

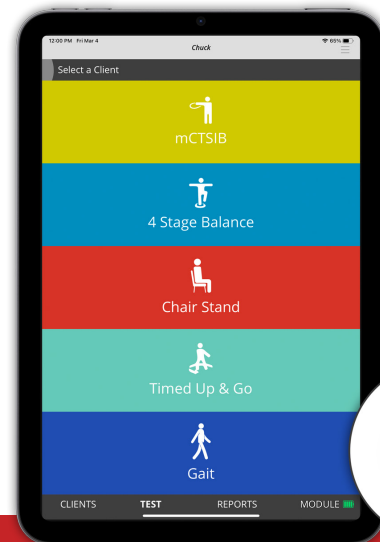

1 Improve documentation & Generate Revenue by increasing assessments with detailed reports

2 Save Time and Standardize Testing with straightforward operation and automated record keeping.

3 Assess and Monitor Client's Balance using evidence-based assessments that highlight fall risk.

- Assess balance, mobility, and fall risk
- Built using evidence-based tests with CDC Steadi Program protocols
- Reporting that's straightforward, comprehensive, and billing-ready
- Portable design: no dedicated space required, easily transported, no computer required
- Intuitive, straightforward, and easy-to-use software and design

Full turnkey solution. Includes tablet, wireless sensor, carrying case, and a full year of software updates.

SENSEN[™]
by
sen
METRIC

Contact for more information
TruMedical[™]
SOLUTIONS

Matt Audas
Cell: 813-493-7779
maudas@tru-medical.com
www.tru-medical.com